

REF	586-2	6×8mL	EXP	2023-12-13	LOT	37E8095	更新日期：	2024-05
1 / 2								

批号：37E8095					
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
(AMY)淀粉酶					
Abbott ARCHITECT c series BSBE \ OTHER	U/L	210	2.5%	18	396
Beckman AU series Beckman Coulter \ OTHER	U/L	215	2.5%	12	670
Roche Hitachi/Hitachi Systems DiaSys \ PNP-G7	U/L	171	1.3%	11	246
Roche cobas 4000/6000/8000 Roche \ PNP-G7	U/L	193	9.9%	<5	<100
Beckman AU series BSBE \ PNP-G5	U/L	201	10.4%	<5	<100
(Ca)钙					
Beckman AU series Reebio \ Arsenazo III	mmol/L	2.92	1.8%	12	673
Roche Hitachi/Hitachi Systems DiaSys \ Arsenazo III	mmol/L	2.66	1.9%	11	228
Roche cobas 4000/6000/8000 Roche \ BAPTA	mmol/L	2.73	10.6%	<5	<100
(Cl)氯					
Roche Hitachi/Hitachi Systems Hitachi \ ISE (indirect)	mmol/L	173	1.7%	11	211
Roche cobas 4000/6000/8000 Roche \ ISE (indirect)	mmol/L	164	10%	<5	<100
(Cortisol)皮质醇					
Roche cobas E601/602 Roche \ Chemiluminescence Immunoassay (CLIA)	ng/mL	297	9.8%	<5	<100
(Cr)肌酐					
Beckman AU series Dedicated \ OTHER	μmol/L	17193	3.7%	19	512
Beckman AU series Reebio \ OTHER	μmol/L	13842	2.3%	12	665
Roche Hitachi/Hitachi Systems DiaSys \ Enzymatic	μmol/L	17607	1.7%	11	245
Roche cobas 4000/6000/8000 Roche \ Enzymatic	μmol/L	15665	10%	<5	<100
(GLU)葡萄糖					
Beckman AU series Reebio \ OTHER	mmol/L	15.9	1.7%	12	677
Roche Hitachi/Hitachi Systems DiaSys \ Glucose oxidase (GOD)	mmol/L	14.3	1.4%	11	223
Roche cobas 4000/6000/8000 Roche \ Hexokinase	mmol/L	14.3	10.4%	<5	<100
(K)钾					
Beckman AU series Beckman Coulter \ ISE (indirect)	mmol/L	59.8	8.9%	12	690
Roche Hitachi/Hitachi Systems Hitachi \ ISE (indirect)	mmol/L	73.6	3.5%	11	241
Roche cobas 4000/6000/8000 Roche \ ISE (indirect)	mmol/L	80.6	10%	<5	<100
Beckman AU series BSBE \ ISE (indirect)	mmol/L	66.3	10%	<5	<100
(MA)微量白蛋白					
Roche cobas c 501/502/701/702 Roche \ OTHER	mg/L	56.6	6.3%	<5	<100
Beckman AU series Dedicated \ Immunoturbidimetry (ITA)	mg/L	61.7	4.7%	19	509
Roche cobas 4000/6000/8000 Roche \ Bromocresol green	mg/L	64.8	9.9%	<5	<100
Beckman AU series BSBE \ Immunoturbidimetry (ITA)	mg/L	78.4	9.9%	<5	<100
(Mg)镁					
Beckman AU series Beckman Coulter \ Xylidyl Blue	mmol/L	1.76	11.9%	<5	<100
(Na)钠					
Beckman AU series Beckman Coulter \ ISE (indirect)	mmol/L	168	1.3%	12	668
Roche Hitachi/Hitachi Systems Hitachi \ ISE (indirect)	mmol/L	168	1%	11	240
Beckman AU series BSBE \ ISE (indirect)	mmol/L	162	7.4%	<5	<100
(P)磷					
Roche Hitachi/Hitachi Systems DiaSys \ Phosphomolybdate-UV-end point	mmol/L	14.2	1.3%	11	244
Roche cobas 4000/6000/8000 Roche \ Phosphomolybdate UV-rate	mmol/L	15.1	10%	<5	<100
(PRO/ TP)(总) 蛋白					
Roche cobas c 501/502/701/702 Roche \ OTHER	g/L	0.595	2%	<5	<100
Abbott ARCHITECT c series BSBE \ OTHER	g/L	0.633	5.1%	17	399
Roche cobas 4000/6000/8000 Roche \ Biuret-endpoint	g/L	0.53	11%	<5	<100

REF	586-2	6×8mL	EXP	2023-12-13	LOT	37E8095	更新日期：	2024-05
2 / 2								

		批号：37E8095			
项目\仪器\试剂\方法	单位	均值 \ CV		Lab \ 点数	
(UA)尿酸					
Roche Hitachi/Hitachi Systems DiaSys \ Uricase colorimetry	μmol/L	857	1.5%	11	245
Roche cobas 4000/6000/8000 Roche \ Uricase colorimetry	μmol/L	813	10%	<5	<100
(Urea)尿素					
Beckman AU series Reebio \ OTHER	mmol/L	262	2.4%	12	675
Roche cobas 4000/6000/8000 Roche \ Urease colorimetry	mmol/L	286	9.8%	<5	<100

注：此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来，仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。