



REF	662-1	6×4.5mL	EXP	2024-05-17	LOT	3F25232	更新日期：	2024-03
1 / 3								

本报告含以下项目的示范实验室数据

参数 (Parameters)	参数 (Parameters)	参数 (Parameters)
(BASO %)嗜碱性粒细胞 %	(EOS %)嗜酸性粒细胞 %	(HCT)血细胞比容/压积
(HGB/ Hb)血红蛋白	(LYM %)淋巴细胞 %	(MCH)平均红细胞血红蛋白含量
(MCHC)平均红细胞血红蛋白浓度	(MCV)平均红细胞体积	(MONO %)单核细胞 %
(NEUT %)嗜中性粒细胞 %	(PLT)血小板	(RBC)红细胞
(WBC)白细胞		

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。



REF	662-1	6×4.5mL	EXP	2024-05-17	LOT	3F25232	更新日期：	2024-03
2 / 3								

批号：3F25232			
项目\仪器\试剂方法	单位	均值	+ / - 2 SD
(BASO %)嗜碱性粒细胞 %			
Sysmex XN series			
sysmex \ Calculate	%	5	0-10
Sysmex XS series			
sysmex \ Calculate	%	10	0-20
Sysmex XT series			
sysmex \ Calculate	%	50	30-70
Sysmex XE series			
sysmex \ Calculate	%	50	30-70
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	5	0-10
(EOS %)嗜酸性粒细胞 %			
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	20	0-40
Sysmex XE series			
sysmex \ Calculate	%	15	0-30
Sysmex XT series			
sysmex \ Calculate	%	15	0-30
Sysmex XS series			
sysmex \ Calculate	%	15	0-30
Sysmex XN series			
sysmex \ Calculate	%	20	0-40
(HCT)血细胞比容/压积			
Sysmex XN series			
sysmex \ Light scattering	%	16.6	13-20.2
Sysmex XT series			
sysmex \ Light scattering	%	16.5	12.9-20.1
Sysmex XS series			
sysmex \ Light scattering	%	16.4	12.8-20
Sysmex XE series			
sysmex \ Light scattering	%	16.6	13.2-20
Mindray BC-6000 series			
Mindray \ Light scattering	%	16.4	12.8-20
(HGB/ Hb)血红蛋白			
Mindray BC-6000 series			
Mindray \ Colorimetry	g/L	60	47.8-72.2
Sysmex XE series			
sysmex \ Light scattering	g/L	60.6	48.4-72.8
Sysmex XS series			
sysmex \ Light scattering	g/L	60.2	47.8-72.6
Sysmex XT series			
sysmex \ Light scattering	g/L	59.9	47.7-72.1
Sysmex XN series			
sysmex \ Light scattering	g/L	60.3	47.9-72.7
(LYM %)淋巴细胞 %			
Sysmex XN series			
sysmex \ Calculate	%	15	5-25
Sysmex XS series			
sysmex \ Calculate	%	15	5-25
Sysmex XT series			
sysmex \ Calculate	%	15	0-30
Sysmex XE series			
sysmex \ Calculate	%	10	0-20
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	15	0-30
(MCH)平均红细胞血红蛋白含量			
Sysmex XN series			
sysmex \ Light scattering	pg	23.6	18.8-28.4
Sysmex XT series			
sysmex \ Light scattering	pg	23.8	19.2-28.4
Sysmex XS series			
sysmex \ Light scattering	pg	23.6	19-28.2
Sysmex XE series			
sysmex \ Light scattering	pg	23.3	18.5-28.1
Mindray BC-6000 series			
Mindray \ Light scattering	pg	23.4	18.6-28.2
(MCHC)平均红细胞血红蛋白浓度			
Mindray BC-6000 series			
Mindray \ Light scattering	g/L	343	281-405
Sysmex XE series			
sysmex \ Light scattering	g/L	340	278-402
Sysmex XS series			
sysmex \ Light scattering	g/L	338	276-400
Sysmex XT series			
sysmex \ Light scattering	g/L	342	278-406
Sysmex XN series			
sysmex \ Light scattering	g/L	339	275-403
(MCV)平均红细胞体积			
Mindray BC-6000 series			
Mindray \ Light scattering	fL	80.5	64.5-96.5
Sysmex XE series			
sysmex \ Light scattering	fL	82.5	66.5-98.5

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。



REF	662-1	6×4.5mL	EXP	2024-05-17	LOT	3F25232	更新日期：	2024-03
3 / 3								

Sysmex XS series sysmex \ Light scattering	fL	84.4	68.4-100
Sysmex XT series sysmex \ Light scattering	fL	81.5	65.5-97.5
Sysmex XN series sysmex \ Light scattering	fL	84.4	68.4-100
(MONO %)单核细胞 %			
Mindray BC-6000 series Mindray \ Flow cytometry	%	5	0-10
Sysmex XE series sysmex \ Calculate	%	5	0-10
Sysmex XT series sysmex \ Calculate	%	10	0-20
Sysmex XS series sysmex \ Calculate	%	10	0-20
Sysmex XN series sysmex \ Calculate	%	5	0-10
(NEUT %)嗜中性粒细胞 %			
Sysmex XN series sysmex \ Calculate	%	50	30-70
Sysmex XS series sysmex \ Calculate	%	60	40-80
Sysmex XT series sysmex \ Calculate	%	50	30-70
Sysmex XE series sysmex \ Calculate	%	60	40-80
Mindray BC-6000 series Mindray \ Flow cytometry	%	55	35-75
(PLT)血小板			
Mindray BC-6000 series Mindray \ Electrical impedance	Giga/L (1E +9/L)	50	30-70
Sysmex XE series sysmex \ Electrical impedance	Giga/L (1E +9/L)	55	35-75
Sysmex XS series sysmex \ Electrical impedance	Giga/L (1E +9/L)	55	35-75
Sysmex XT series sysmex \ Light scattering	Giga/L (1E +9/L)	55	35-75
Sysmex XN series sysmex \ Light scattering	Giga/L (1E +9/L)	50	30-70
(RBC)红细胞			
Sysmex XN series sysmex \ Light scattering	T/L (1E +12/L)	2.29	1.81-2.77
Sysmex XT series sysmex \ Light scattering	T/L (1E +12/L)	2.32	1.84-2.8
Sysmex XS series sysmex \ Light scattering	T/L (1E +12/L)	2.29	1.83-2.75
Sysmex XE series sysmex \ Light scattering	T/L (1E +12/L)	2.31	1.85-2.77
Mindray BC-6000 series Mindray \ Electrical impedance	T/L (1E +12/L)	2.29	1.81-2.77
(WBC)白细胞			
Mindray BC-6000 series Mindray \ Electrical impedance	Giga/L (1E +9/L)	3.14	2.48-3.8
Sysmex XE series sysmex \ Light scattering	Giga/L (1E +9/L)	3.13	2.47-3.79
Sysmex XS series sysmex \ Light scattering	Giga/L (1E +9/L)	3.16	2.48-3.84
Sysmex XT series sysmex \ Light scattering	Giga/L (1E +9/L)	3.14	2.48-3.8
Sysmex XN series sysmex \ Light scattering	Giga/L (1E +9/L)	3.14	2.48-3.8

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的变化，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。