



REF	662-3	6×4.5mL	EXP	2024-05-17	LOT	3F2A052	更新日期:	2024-03
								1 / 3

本报告含以下项目的示范实验室数据

参数 (Parameters)	参数 (Parameters)	参数 (Parameters)
(BASO %)嗜碱性粒细胞 %	(EOS %)嗜酸性粒细胞 %	(HCT)血细胞比容/压积
(HGB/ Hb)血红蛋白	(LYM %)淋巴细胞 %	(MCH)平均红细胞血红蛋白含量
(MCHC)平均红细胞血红蛋白浓度	(MCV)平均红细胞体积	(MONO %)单核细胞 %
(NEUT %)嗜中性粒细胞 %	(PLT)血小板	(RBC)红细胞
(WBC)白细胞		

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。



REF	662-3	6×4.5mL	EXP	2024-05-17	LOT	3F2A052	更新日期： 2024-03
							2 / 3

批号： 3F2A052			
项目\仪器\试剂方法	单位	均值	+ / - 2 SD
(BASO %)嗜碱性粒细胞 %			
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	5	0-10
Sysmex XE series			
sysmex \ Calculate	%	50	30-70
Sysmex XS series			
sysmex \ Calculate	%	10	0-20
Sysmex XT series			
sysmex \ Calculate	%	15	0-30
Sysmex XN series			
sysmex \ Calculate	%	10	0-20
(EOS %)嗜酸性粒细胞 %			
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	15	5-25
Sysmex XE series			
sysmex \ Calculate	%	15	0-30
Sysmex XS series			
sysmex \ Calculate	%	15	5-25
Sysmex XT series			
sysmex \ Calculate	%	15	0-30
Sysmex XN series			
sysmex \ Calculate	%	20	0-40
(HCT)血细胞比容/压积			
Mindray BC-6000 series			
Mindray \ Light scattering	%	54.5	44.1-64.9
Sysmex XE series			
sysmex \ Light scattering	%	54.2	43.8-64.6
Sysmex XS series			
sysmex \ Light scattering	%	53.9	43.5-64.3
Sysmex XT series			
sysmex \ Light scattering	%	54.1	43.5-64.7
Sysmex XN series			
sysmex \ Light scattering	%	54.4	44.2-64.6
(HGB/ Hb)血红蛋白			
Mindray BC-6000 series			
Mindray \ Colorimetry	g/L	189	153-225
Sysmex XE series			
sysmex \ Light scattering	g/L	187	151-223
Sysmex XS series			
sysmex \ Light scattering	g/L	186	152-220
Sysmex XT series			
sysmex \ Light scattering	g/L	189	153-225
Sysmex XN series			
sysmex \ Light scattering	g/L	186	150-222
(LYM %)淋巴细胞 %			
Mindray BC-6000 series			
Mindray \ Flow cytometry	%	20	10-30
Sysmex XE series			
sysmex \ Calculate	%	25	10-40
Sysmex XS series			
sysmex \ Calculate	%	20	10-30
Sysmex XT series			
sysmex \ Calculate	%	25	10-40
Sysmex XN series			
sysmex \ Calculate	%	25	10-40
(MCH)平均红细胞血红蛋白含量			
Mindray BC-6000 series			
Mindray \ Light scattering	pg	34	27.8-40.2
Sysmex XE series			
sysmex \ Light scattering	pg	32.9	26.7-39.1
Sysmex XS series			
sysmex \ Light scattering	pg	33	26.8-39.2
Sysmex XT series			
sysmex \ Light scattering	pg	33.1	26.7-39.5
Sysmex XN series			
sysmex \ Light scattering	pg	33.1	26.7-39.5
(MCHC)平均红细胞血红蛋白浓度			
Mindray BC-6000 series			
Mindray \ Light scattering	g/L	331	267-395
Sysmex XE series			
sysmex \ Light scattering	g/L	328	264-392
Sysmex XS series			
sysmex \ Light scattering	g/L	328	264-392
Sysmex XT series			
sysmex \ Light scattering	g/L	327	263-391
Sysmex XN series			
sysmex \ Light scattering	g/L	330	264-396
(MCV)平均红细胞体积			
Mindray BC-6000 series			
Mindray \ Light scattering	fL	96	78-114
Sysmex XE series			
sysmex \ Light scattering	fL	97	79-115

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。



REF	662-3	6×4.5mL	EXP	2024-05-17	LOT	3F2A052	更新日期：	2024-03
3 / 3								

Sysmex XS series sysmex \ Light scattering	fL	97	79-115
Sysmex XT series sysmex \ Light scattering	fL	94	76-112
Sysmex XN series sysmex \ Light scattering	fL	96	78-114
(MONO %)单核细胞 %			
Mindray BC-6000 series Mindray \ Flow cytometry	%	5	0-10
Sysmex XE series sysmex \ Calculate	%	10	0-20
Sysmex XS series sysmex \ Calculate	%	10	0-20
Sysmex XT series sysmex \ Calculate	%	5	0-10
Sysmex XN series sysmex \ Calculate	%	5	0-10
(NEUT %)嗜中性粒细胞 %			
Mindray BC-6000 series Mindray \ Flow cytometry	%	60	40-80
Sysmex XE series sysmex \ Calculate	%	50	30-70
Sysmex XS series sysmex \ Calculate	%	50	30-70
Sysmex XT series sysmex \ Calculate	%	60	40-80
Sysmex XN series sysmex \ Calculate	%	50	30-70
(PLT)血小板			
Mindray BC-6000 series Mindray \ Electrical impedance	Giga/L (1E +9/L)	450	360-540
Sysmex XE series sysmex \ Electrical impedance	Giga/L (1E +9/L)	450	350-550
Sysmex XS series sysmex \ Electrical impedance	Giga/L (1E +9/L)	455	345-565
Sysmex XT series sysmex \ Light scattering	Giga/L (1E +9/L)	455	355-555
Sysmex XN series sysmex \ Light scattering	Giga/L (1E +9/L)	450	350-550
(RBC)红细胞			
Mindray BC-6000 series Mindray \ Electrical impedance	T/L (1E +12/L)	5.49	4.43-6.55
Sysmex XE series sysmex \ Light scattering	T/L (1E +12/L)	5.52	4.46-6.58
Sysmex XS series sysmex \ Light scattering	T/L (1E +12/L)	5.56	4.5-6.62
Sysmex XT series sysmex \ Light scattering	T/L (1E +12/L)	5.53	4.41-6.65
Sysmex XN series sysmex \ Light scattering	T/L (1E +12/L)	5.53	4.41-6.65
(WBC)白细胞			
Mindray BC-6000 series Mindray \ Electrical impedance	Giga/L (1E +9/L)	17.7	14.1-21.3
Sysmex XE series sysmex \ Light scattering	Giga/L (1E +9/L)	17.8	14.2-21.4
Sysmex XS series sysmex \ Light scattering	Giga/L (1E +9/L)	17.9	14.3-21.5
Sysmex XT series sysmex \ Light scattering	Giga/L (1E +9/L)	17.9	14.3-21.5
Sysmex XN series sysmex \ Light scattering	Giga/L (1E +9/L)	18	14.4-21.6

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。